# LIGADON®

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LIGAMENT  
REINFORCEMENT  
FOR  
PERCUTANEOUS  
TENOSYNTHESIS  
OF ACHILLES  
TENDON

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● The therapeutic method used for severed Achilles tendon usually oscillates between two well-known methods:

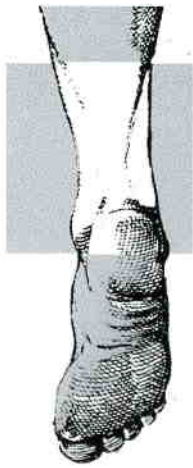
- *Non operative orthopaedic treatment*
- *Surgical treatment.*

*In-between these two solutions, the new technique of percutaneous tenosynthesis of Achilles tendon to avoid drawbacks and risks of these methods.*

*The design of this technique has been to simplify the surgery and to offer a minimal assault.*

● As a matter of fact, the main principals and advantages of this technique are :


- *Joining and retention of two flanking sinews by percutaneous suture*
- *Care for the sunovial sleeve around the broken zone in order to avoid devascularisation and necrosis*
- *Suppression of the risks of cutaneous necrosis and disconnection*
- *A shorten time of hospitalisation.*

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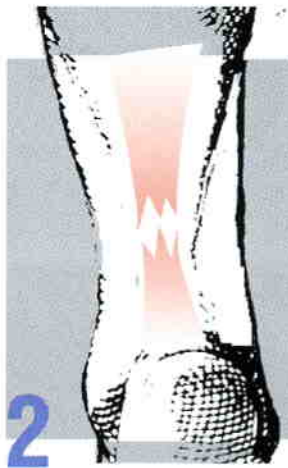*Installation in prone position, foot outside the table in order to freely manipulate the tibio-tarsal in flexion-extension.*

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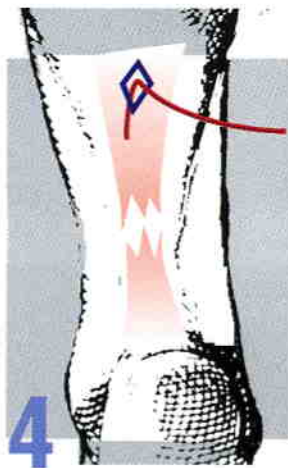
*Identifying the rupture:* Identifying the rupture area by palpation, marking it eventually with a demographic pencil. Bending the needle according to the required curve.

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
*Upper Incision :* It is located 6cm up from the rupture and 2cm length along the internal side of the tendon to avoid the external sural nerve.

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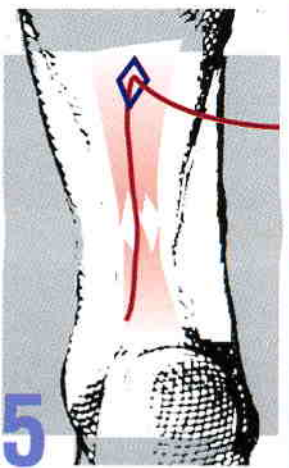
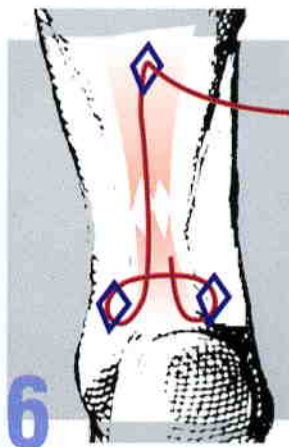
*Way of the needle :* Introduce of the needle into the thickness of the tendon and guide it along progress of the needle may be checked continuously by palpating the tendon.

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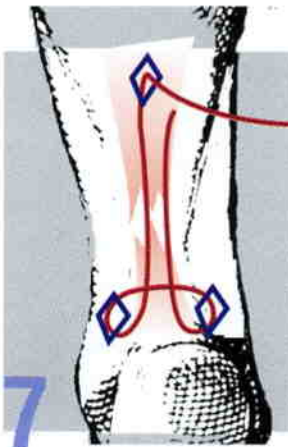
*Distal frame:* Make a stitch "en cadre" on the lower fragment by two "little incisions" in this particularly difficult cicatrization area after checking the joining of the sinews and the good tension of the ligament.

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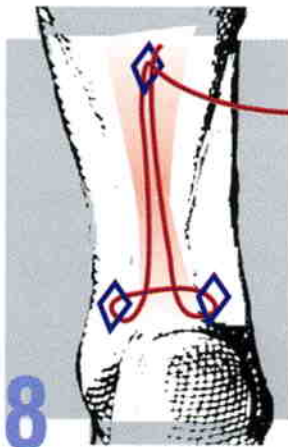
*Way back of the needle:* Stitch up into the thickness of the tendon up to the upper frame.

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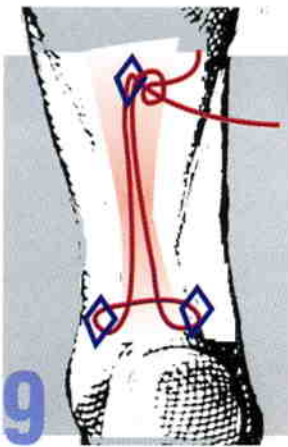*Putting into tension:* Put in equine the foot up to the joining of the fibres of the severed tendon under palpation control.

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*Upper knot :* Make the final knot by taking fulcrum or not on the patch provided for this purpose. The importance of the convex side of this last item is to allow homogenous pressure. One could avoid use of this patch and knot directly the thread. Thanks to its large diameter, it is easy to stitch into the thread to facilitate the knotting process.